

# OXYGEN INFUSION GUN



**Hydrating and Calming Oxygen Infusion performed with the Oxygen serum of your choice.**

This step provides deep skin hydration with an intense flow of Oxy Serums to stimulate skin turnover, boost collagen production, increase skin elasticity, and infuse skin with minerals and vitamins.

- 1. Z-LuminOxy Skin Brightening & Revitalizing Serum**
- 2. Z-RevitaOx Revitalizing Antioxidant Solution**
- 3. Z-OxyTight Skin Repairing & Boosting Serum**
- 4. Zemits VitaLuster Vit C Serum**

## **Oxygen Treatment Protocol:**

1. Once you've selected your treatment serum, add 2-3 ml to the oxygen gun.
2. Select Oxygen mode using the 'select' button. Press start.
3. Use the trigger to control how much serum is released during the treatment.
4. Hold the oxygen gun right above the skin and slowly release serum, starting from the chin and working from the center of the face out towards the periphery.
5. If extra serum is left, treat the client's neck and décolleté area (optional).

